



## Rim Former Sterilization Instructions

### **Chemical sterilization:**

The Blue Dolphin Rim former is not intended for patient contact. It is considered a non critical item and as such can be sterilized by immersing in an EPA registered hospital disinfectant, following the manufacturers recommendations.

### **AutoClave Sterilization:**

First remove any residual wax from the instrument by heating over a bunsen burner and wiping clean with a paper towel or disposable cloth.

Place the rim former by itself in a protective pouch or wrap to prevent any remaining wax on the instrument from contaminating the autoclave or other instruments.

Sterilize in the autoclave at 250°F for 15-20 minutes at 15 psi.

Follow standard procedures for cooling and removal.